Instructions for use Breastmilk Bottles

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EN Breastmilk Bottles

Ideal to collect, store and freeze breastmilk

Cleaning and Hygiene

Important

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.

Prior to first use and once daily

Disassemble, wash and rinse all parts. Place parts in a saucepan of sufficient capacity. Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale. Heat it up and allow to boil for 5 minutes.

Alternatively, the disassembled parts can be sanitized in Medela Quick Clean Microwave Bags or in top rack of dishwasher or steam-sterilized.

After each use

Disassemble and wash all parts in warm, soapy water. Rinse in clear water and allow parts to air-dry on a clean towel.

Warning

- Keep all components not in use out of the reach of children.

Caution

- Avoid touching the inside of bottles and lids after cleaning.

Storing / thawing breastmilk

(For Healthy Term Babies)

Freshly expressed breastmilk	
Room Temperature	Do not leave at room temperature
Refrigerator	3 – 5 days at approx. 4° C
Refrigerator Freezer Compartment	6 months at approx. –16° C
Deep Freezer	12 months at approx. –18° C
Thawed breastmilk (previously frozen)	
Room Temperature	Do not store
Refrigerator	10 hours
Refrigerator Freezer Compartment	Never refreeze
Deep Freezer	Never refreeze

- When freezing do not fill bottles more than 3/4 full to allow space for expansion.
- Label bottles with the date of pumping.
- Do not store breastmilk in the door of the refrigerator.
- In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle under running warm water (max. 37° C).
- Gently shake the bottle to blend any fat that has separated.
- Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).

Material:	Polypropylene
Disposal:	According to local regulations

Does not contain Bisphenol-A, DEHP (Phthalate) or Latex. Conforms to EN 14350.

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