Instructions for use

Quick Clean™

Microsteam Bags

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Quick Clean™

Mikrowellen-Beutel

Mode d'emploi

Quick Clean™

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


Quick Clean™

Sacche per Microonde

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Quick Clean™

Magnetronzakken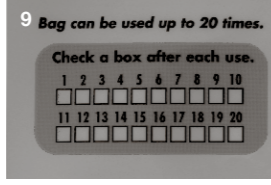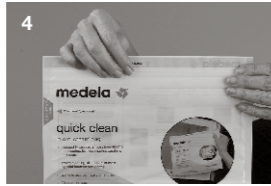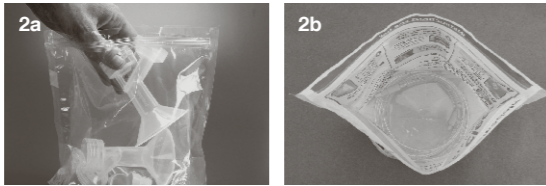

EN

Quick Clean™

Microsteam Bags

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Follow these cleaning instructions.

One bag will hold

- 1 breastshield, connector, valve, membrane, milk bottle and cap or
- 2 breastshields, connectors, valves and membranes or
- 2 milk bottles and caps or
- Breastpump tubing

- Disassemble and thoroughly wash all parts in warm, soapy water. Rinse in clear water.

<ol style="list-style-type: none"> Put either <ul style="list-style-type: none"> 1 breastshield, connector, valve, membrane, milk bottle and cap or 2 breastshields, connectors, valves and membranes or 2 milk bottles and caps 	<ol style="list-style-type: none"> Lay breastpump tubing flat on bottom of bag. Do not place any other items in the bag with the tubing.
<ol style="list-style-type: none"> For breastshields, connectors, valves and membranes, milk bottles: Add 60ml water to bag. 	<ol style="list-style-type: none"> For breastpump tubing: Add 120 ml water to bag. Make sure tubing is covered with water.

- Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- Do not overfill bag.
- Using less than 60 ml/120 ml water may warp the parts.

- Seal bag tightly. Put bag into microwave on center of turntable. Do not place the bag on the metallic grill that comes with some microwave ovens.

- Heat in microwave for:

1.5 minutes at 1100 W+	3 minutes at 800 - 1100 W	5 minutes at 500 - 750 W
------------------------	---------------------------	--------------------------

Heating for more than the recommended time may disfigure plastic parts. If your microwave does not have a turntable, rotate bag halfway through.

- Using oven mitts or hot pad, carefully remove bag from microwave oven.

- Before opening bag, remove water by pouring out through steam vent.

Caution
When removing bag from microwave or opening bag, hot water or steam may escape (danger of burning).

- Remove parts from bag and dry them with a clean towel. After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid. Alternatively, the parts can be wrapped in a clean paper or cloth towel.

- Bag can be used up to 20 times. Check a box after each use.

Safeguards

- Bag intended for use with microwave-safe plastics only.
- Piston and cylinder of Medela Manual Breastpump cannot be cleaned in microwave.

Note

- Steam cleaning may discolor components or may cause clouding in the tubing. This will not affect performance.
- Do not place the bag in a conventional oven or in an oven grill.
- Follow the instructions for your microwave oven.

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